Capillary Blood Collection Instructions

Please read all instructions **before starting**. Follow each step carefully to ensure your sample is successfully collected and returned.

PREPARE FOR COLLECTION

Place the cold pack in a freezer for at least **30 minutes before sample return**. The cold pack is required to maintain appropriate sample temperature during shipment.

CHECK AND LABEL THE TUBE

You will use a **purple-capped tube** to collect your sample.

If the tube is pre-labeled:

- **Confirm the name and date of birth** on the tube are correct **before** collecting your sample.

If the tube is not pre-labeled:

- Write your **full name** (as used on medical records) in pen on the label provided.
- Apply the label to the tube after writing. **Do not cover the barcode or the fill line on the tube.**

COLLECT YOUR SAMPLE

- Please **follow the enclosed Tasso+® instructions** for proper collection.
- If the tube does not fill to the top line, **return the sample** with the amount collected.

Scan here for an instructional video:

Package & Return the Sample

Please read through and follow each of these steps to ensure your sample is successfully returned.

1. Secure the tube cap by pressing firmly until it **clicks**.
2. Place the tube in the provided biohazard bag with the absorbent pad and seal the bag.
3. Place the sealed bag in the insulated pouch with the frozen cold pack.
4. Place the pouch in the shipping box.
5. **Write the collection date** (MM/DD/YY) on the label in the box.
6. Attach the prepaid return shipping label to the back of the box.
7. Close and seal the box using the adhesive strip.
8. Drop off the package with the listed carrier **within one day of collection**.

Questions?

Contact your study team directly for assistance.

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